



🔍 Use a keyword, test name or number

## Synthetic Opioids, Screen With Reflexed Confirmation, Urine

**TEST: 703025** CPT: 80307

**Test Includes** Synthetic opioid analysis includes the following analytes: Fentanyl, Acetyl fentanyl, Norfentanyl, Acetyl Norfentanyl, Furanyl fentanyl, p-Fluorobutyrylfentanyl, Acrylfentanyl, Butyryl fentanyl, Carfentanil, 3-Methyl fentanyl, Valeryl fentanyl, Methoxyacetyl fentanyl, Cyclopropyl fentanyl, Tetrahydrofuran fentanyl, Phenylacetyl fentanyl, p-Fluoro cyclopropyl fentanyl, MT-45, Butyryl fentanyl carboxy metabolite, Isotonitazine, Metonitazene, Acetylfentanyl-4-methylphenethyl, Ocfentanil, p-Fluorofentanyl, 4-Methoxy-butyryl fentanyl, AH-7921, U-47700, AH-8533, AH-8529, IC-26, B-Hydroxythiofentanyl, Norcarfentanil, Xylazine, 4-Hydroxy xylazine, Methoxyacetyl Norfentanyl, Mitragynine, 7-OH-mitragynine.

**Expected** 5 - 7 days

**Turnaround  
Time**

Turnaround time is defined as the usual number of days from the date of pickup of a specimen for testing to when the result is released to the ordering provider. In some cases, additional time should be allowed for additional confirmatory or additional reflex tests. Testing schedules may vary.

**Related  
Documents**

- [Sample Report](#)

### SPECIMEN REQUIREMENTS

**Specimen** Urine (random)

**Volume** 10 mL

**Minimum  
Volume** 1 mL

Container	Plastic urine container without preservative
Collection	Collect urine in a plastic urine container without preservative.
Storage Instructions	<b>Submission/transport (&lt;3 days):</b> Room temperature. For storage beyond three days, specimen should be refrigerated or frozen.
Causes for Rejection	Urine in preservative container

**TEST DETAILS**

Use	Detect use of synthetic opioid drugs
Methodology	Initial presumptive testing by liquid chromatography tandem mass spectrometry (LC/MS-MS); presumptive positives confirmed by definitive liquid chromatography tandem mass spectrometry (LC/MS-MS).

© 2023 Laboratory Corporation of America® Holdings and Lexi-Comp Inc. All Rights Reserved.

CPT Statement/Profile Statement

The LOINC® codes are copyright © 1994-2023, Regenstrief Institute, Inc. and the Logical Observation Identifiers Names and Codes (LOINC) Committee. Permission is granted in perpetuity, without payment of license fees or royalties, to use, copy, or distribute the LOINC® codes for any commercial or non-commercial purpose, subject to the terms under the license agreement found at <https://loinc.org/license/>. Additional information regarding LOINC® codes can be found at LOINC.org, including the LOINC Manual, which can be downloaded at [LOINC.org/downloads/files/LOINCManual.pdf](https://loinc.org/downloads/files/LOINCManual.pdf)

[Privacy Statement](#)     
 [Terms and Conditions](#)     
 [Notice of Nondiscrimination](#)

[Combatting Modern Slavery and Human Trafficking Statement](#)

© 2023 Laboratory Corporation of America® Holdings. All Rights Reserved.